



## High-Level Disinfection Kit

A 3-step wipe cleaning & disinfecting kit for:

- Endoscopes
- Ultrasound probes
- Semi-critical medical devices

### Product Description

The High Level Disinfection Kit includes 4 different cleaning & disinfecting wipes, to be used in the following 3 simple and effective steps:

#### Step 1

Includes 2 similar individually packed wipes, saturated with Enzymatic Detergent - marked "Step 1" - for the initial cleaning of endoscopes, ultrasound probes and other semi-critical medical devices, in a simple and easy way.

Active ingredients: Detergent + Enzymes.

#### Step 2

Includes 1 individually packed wipe – marked "Step 2" saturated with a High-Level Disinfectant - solution – that thoroughly disinfects the medical device, killing all germs and bacteria.

Contact time: 5 minutes

#### Step 3

includes 1 individually packed sterile wipe – marked "Step 3" – saturated with Deionized water for rinsing the endoscope of all solution remaining after the disinfection process, and prepares it for the next use.

### Includes

The offered kit includes 20 sets packed in a box.

The solution is commonly used as a high-level disinfectant for medical instruments and devices. Disinfection with the solution is indicated for semi-critical medical devices and instruments that come into contact with mucous membranes or broken skin, such as endoscopes, specula, laryngeal mirrors and internal ultrasound probes.

#### Potential benefits include:

- ✓ Lower inhalation exposure risk
- ✓ Reduced disinfecting time
- ✓ Solution is approved for use on almost all MHA equipment without negating the warranty
- ✓ Cost is significantly less than installing a more substantial ventilation system to minimize respiratory irritation from using glutaraldehyde.

The solution has been approved by the FDA for High-Level Disinfection of medical devices.

